

HYALURONIC ACID BIOREVITALIZATION

Opéra
K-surgery

Mariagrazia Croccia, MD PhD

PISA, ITALY

Opéra

Una nota di elasticità per la tua bellezza
An elasticity note for your beauty

K-surgery brand had close to its heart the concept of unicity, both in aesthetic and medical area, in order to be able to satisfy the needs of each costumers.

Mariagrazia Croccia, MD PhD



The approval of this concept helped to create Opéra: a set of professional dermic filler treatments that, thanks to their different hyaluronic acid combinations, allow you to satisfy every type of need. Also the name has been chosen both for its international notoriety, both to highlight the goal of the treatment: to gift and to exalt the beauty of each person.



Opéra is a sterile medical device, injectable, apyrogenic, absorbable, based on cross-linked hyaluronic acid derived from bacterial fermentation. The crosslinking of Hyaluronic Acid inside the product makes the filler effect more stable. The fillers Opéra, thanks to the different concentrations of Hyaluronic Acid put inside in each product, guarantee achievement of more goals depending on each patient needs.

Opéra

Una nota di elasticità per la tua bellezza
An elasticity note for your beauty

REVITALIZATION

WHAT'S NEW?

A revitalizing touch for face

Opéra ^{rev}
Confezione/Pack
1x1ml
Capacità volumizzante* Volumizing effect*
○
PH
7
Ago/Needle
30G

Mariagrazia Croccia, MD PhD

Opéra^{rev}

SKIN BIOREVITALIZATION

Transdermal skin injection of different substances (vitamins, hormones, enzymes, hyaluronic acid, plant extracts) to stimulate fibroblast biosynthetic capacity and facilitate cellular interaction.



Mariagrazia Croccia, MD PhD

Opéra^{rev}

BIOREVITALIZATION: AIM

Increase the production of collagen and elastin.

Maintain or regain a healthy and youthful appearance of the face with firm, hydrated and shiny skin.



Obtain restoration of face volume with reduction of wrinkles.

Opéra^{rev}

INDICATION

Various degrees of skin aging.


Skin dehydration.

Loss of tone and elasticity.

Dull or uneven complexion.

Maintenance of other medical-aesthetic treatments.

Opéra^{rev}

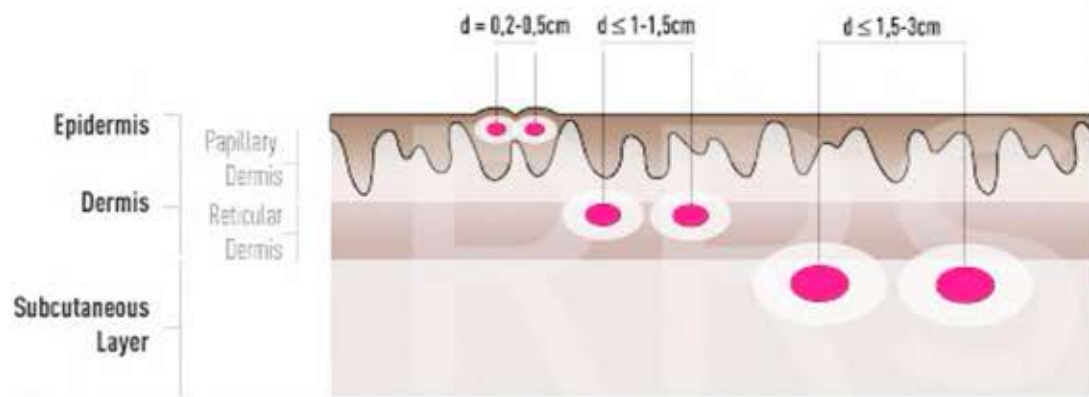

PROCEDURAL TECHNIQUE

intra-epidermal injections (depth 2-5 mm);

dermo-epidermal junction injections
(up to 10-15 mm);

subcutaneous injections (up to 1,5-3 cm)

Opéra^{rev}

OPERA REV: HYALURONIC ACID

Its **weakly cross linked** formulation makes this product suitable for the patients who want a delicate but effective effect.



Its bioadsorbable property makes it ideal for the correction of minor facial skin depressions.

COMPETITIVE SET*

* Italian market

Opéra^{rev}

BAP TECHNIQUE

ANTI AGING

Metodo 1
Method 1

Metodo 2
Method 2

MICRO INJECTIONS TECHNIQUE

PHOTO AND CRONO AGING
DAMAGE

Mariagrazia Croccia, MD PhD



La confezione contiene:

- 1 siringa pre-riempita graduata a 1ml
- due aghi 30G

The pack contains:

- 1 graduated pre-filled syringe for 1 ml
- two 30G needles

Metodo 1 - Method 1



1 seduta
ogni 3 settimane



1 session
every 3 weeks

Metodo 2 - Method 2



3-5 sedute bisettimanali
per 2-3 volte l'anno




3-5 sessions twice a week
for 2-3 times a year

Applicazione:

- anti aging
- per danni da foto e crono aging

Treatment:

- anti aging
- damages from photo and chrono aging

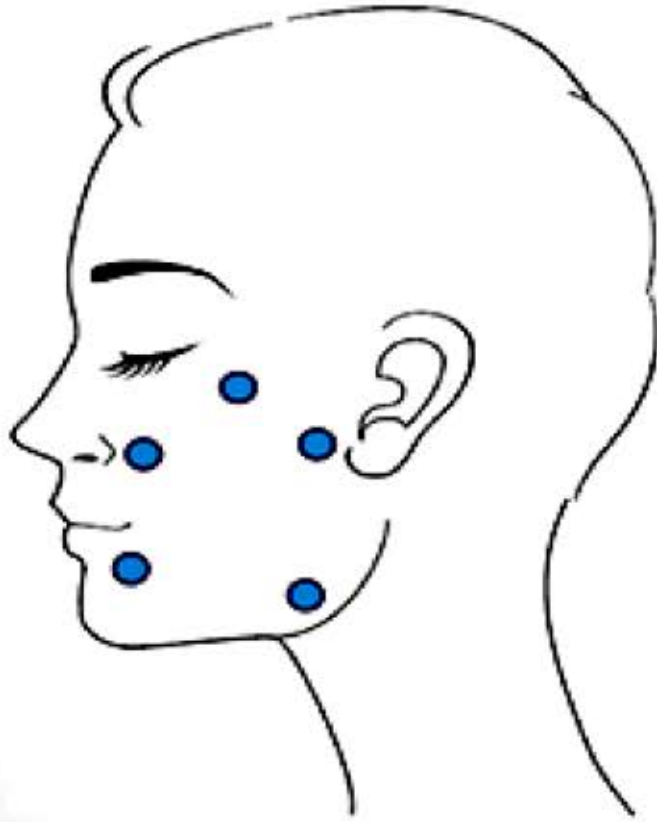

BAP TECHNIQUE (*BIO-AESTHETIC-POINT*):


Full face treatment.

5 injections per half face.


Safe Anatomic points (avoiding large vessels or nerves).

Maximum result with minimum number of injections.


Extremely comfortable fo the patients.

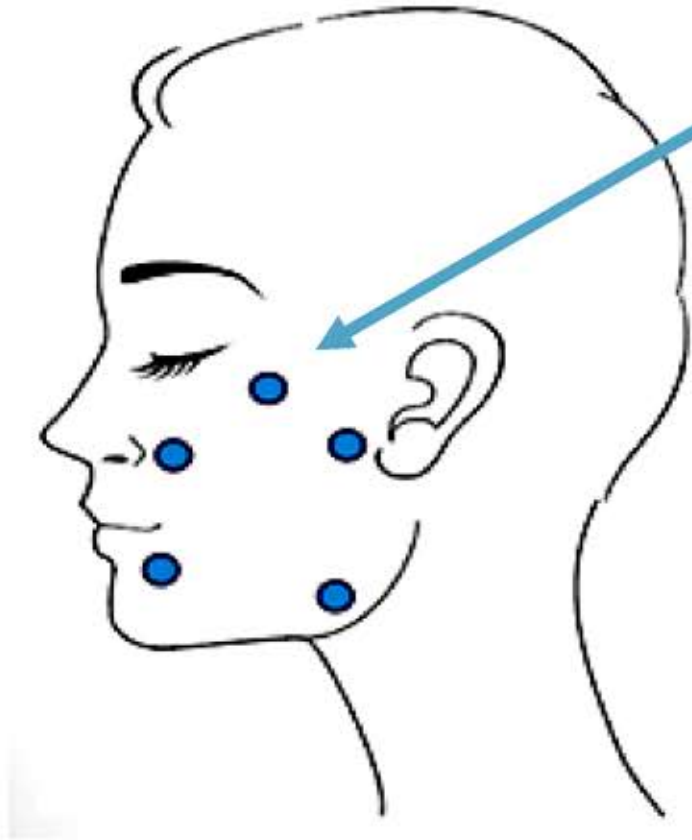


1. on cheekbone protrusion, at least 2 cm from the external eye.

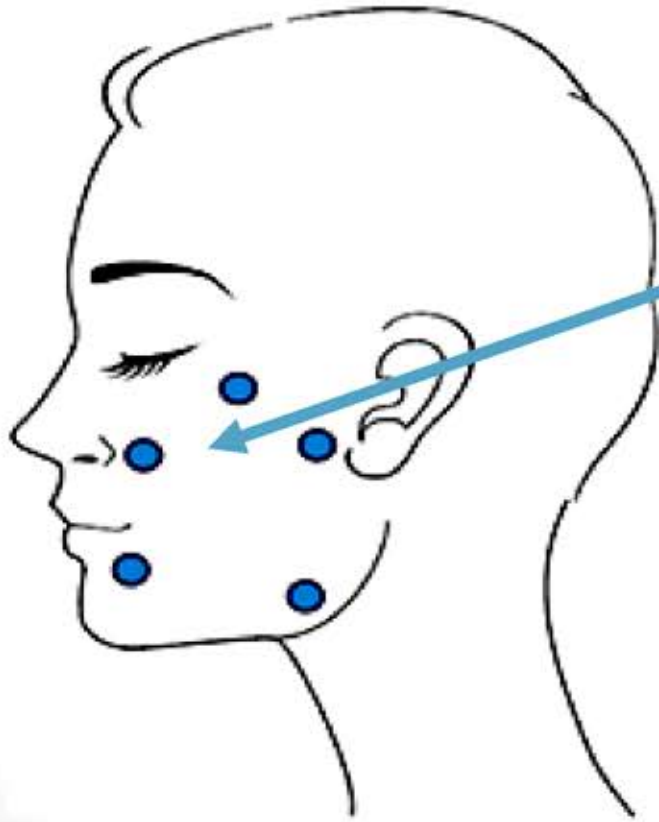


1. on cheekbone protrusion, at least 2 cm from the external eye.
2. at the intersection of the line joining the wing of the nose to the tragus and the line passing through the pupil.

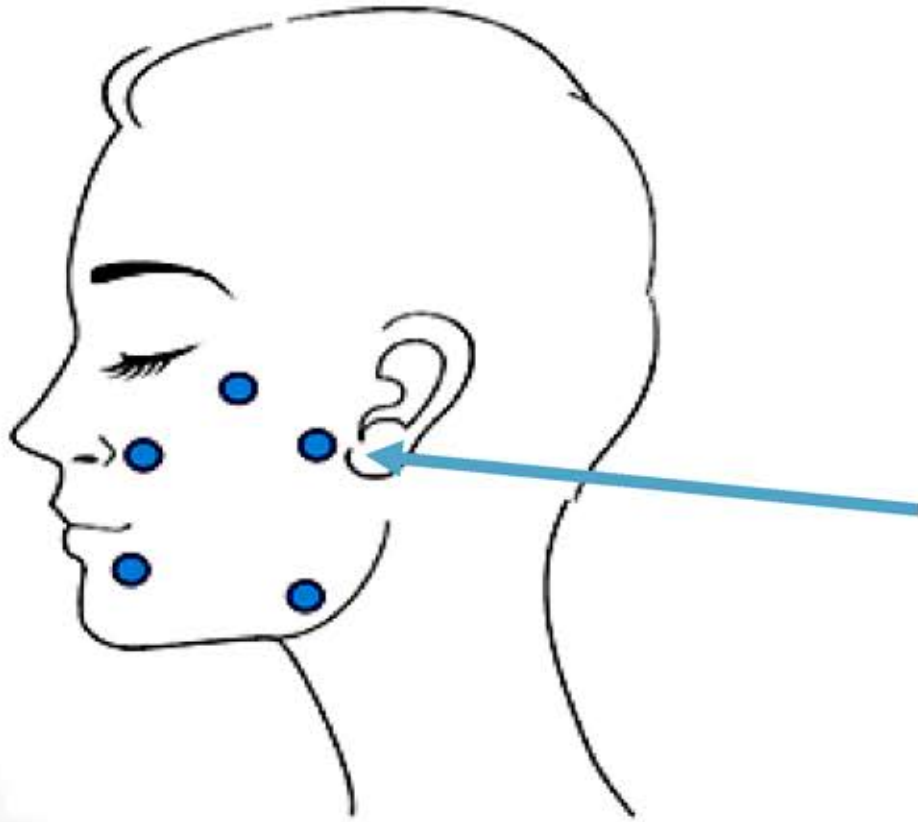

1. on cheekbone protrusion, at least 2 cm from the external eye.
2. at the intersection of the line joining the wing of the nose to the tragus and the line passing through the pupil.
3. 1 cm anteriorly to the tragus.

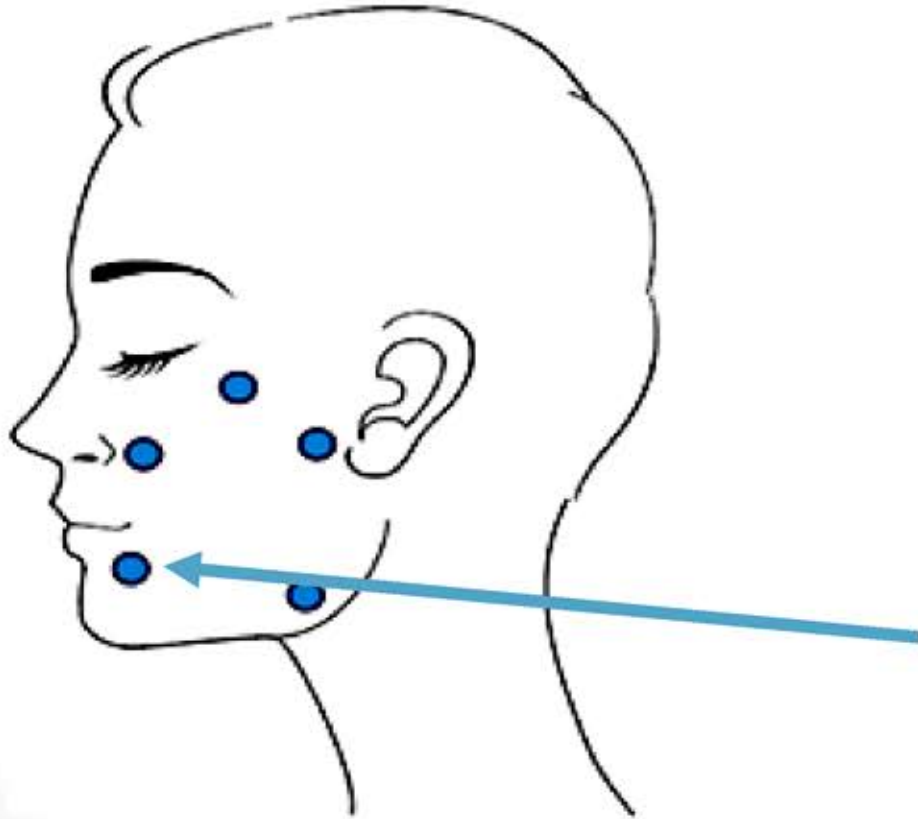

1. on cheekbone protrusion, at least 2 cm from the external eye.
2. at the intersection of the line joining the wing of the nose to the tragus and the line passing through the pupil.
3. 1 cm anteriorly to the tragus.
4. 1,5 cm lateral to the line passing through the center of the chin (at 1/3 of its height).

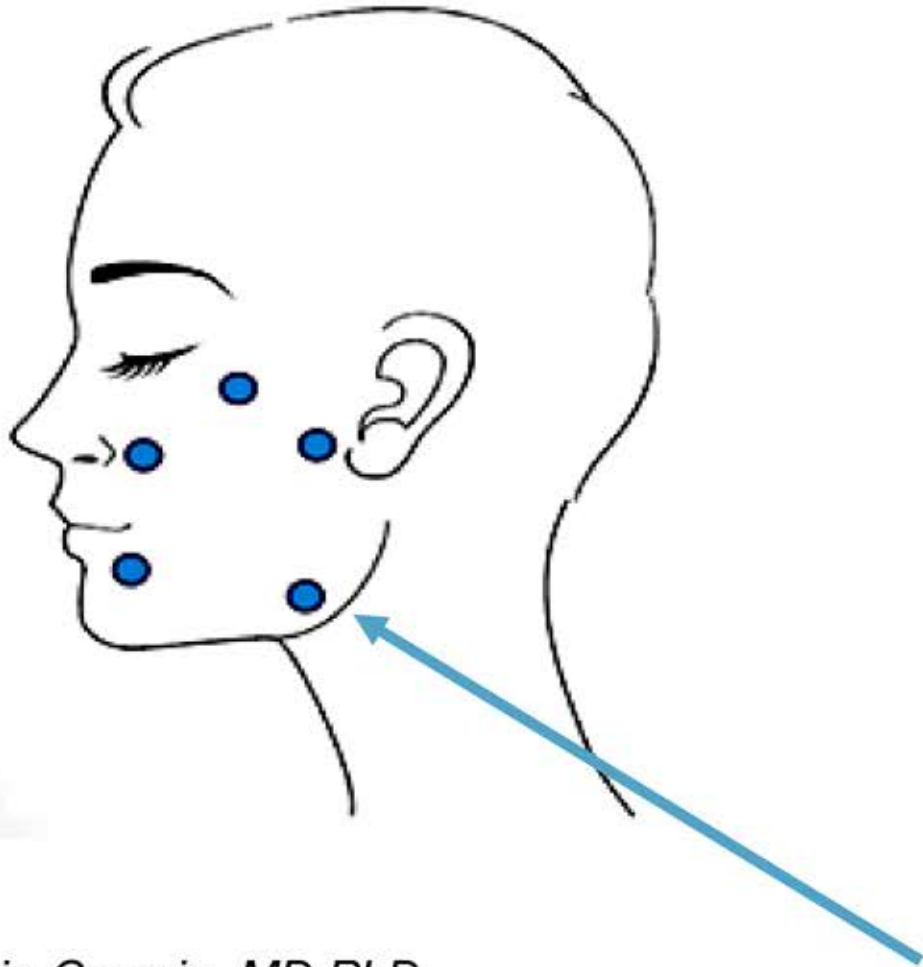

1. on cheekbone protrusion, at least 2 cm from the external eye.
2. at the intersection of the line joining the wing of the nose to the tragus and the line passing through the pupil.
3. 1 cm anteriorly to the tragus.
4. 1,5 cm lateral to the line passing through the center of the chin (at 1/3 of its height).
5. 1 cm above the mandibular angle.

Opéra^{rev}

Confezione/Pack

1x1ml

Capacità volumizzante*
*Volumizing effect**

○

PH

7

Ago/Needle

30G

BAP TECHNIQUE:

1-2 ml x session


0,5-1 ml x half-face

0,1-0,2 ml x BAP

20 mg/ml.

We suggest to use a **30G** needle.

Depth 6-7 mm.

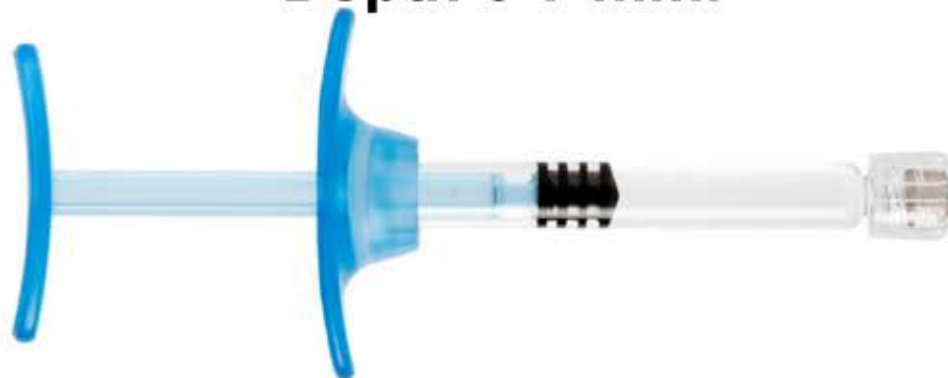



OPERA REV: EFFECTS

Restoration of fibroblasts function to produce new collagen and elastin fibers.

Reconstitution of the extracellular matrix with better hydration.

Enhance of total cellular activity.

Compensation for the hyaluronic acid deficiency.

Opéra^{rev}

RESULTS

Mariagrazia Crocchia, MD PhD

Opéra^{rev}

RESULTS

1 session



Mariagrazia Croccia, MD PhD

Opéra^{rev}

RESULTS

2 sessions

Mariagrazia Croccia, MD PhD



Opéra^{rev}

RESULTS
1 session

Mariagrazia Croccia, MD PhD



Opéra^{rev}

RESULTS

Use on neck? FEASEBLE

**Immediately post-procedural
signs on skinny neck?
NEGLIGEBLE**

Mariagrazia Croccia, MD PhD



Opéra^{rev}



THANK YOU

Mariagrazia Croccia MD PhD

Waiting for you in Italy